Structural Limitations
Limitations Structurelles
Strukturelle Einschränkungen
Limiti strutturali
Limitaciones Estructurales

SmartIP

Swing Control Cylinder

Fitting Instructions

239629

EN	Fitting Instructions	2
FR	Notice de Réglage	6
DE	Montageanleitung	10
IT	Istruzioni di montaggio	14
ES	Instrucciones de ajuste	18

Blatchford:

SMARTIP cylinder attachment to ESK+ Full Shin



(Fitting Kit 019768)

The ESK+ extension stop should be wound fully home. No adjustment is necessary.

User/attendant should be aware of potential finger trap hazards at all times.

SMARTIPSFF SMARTIP4BF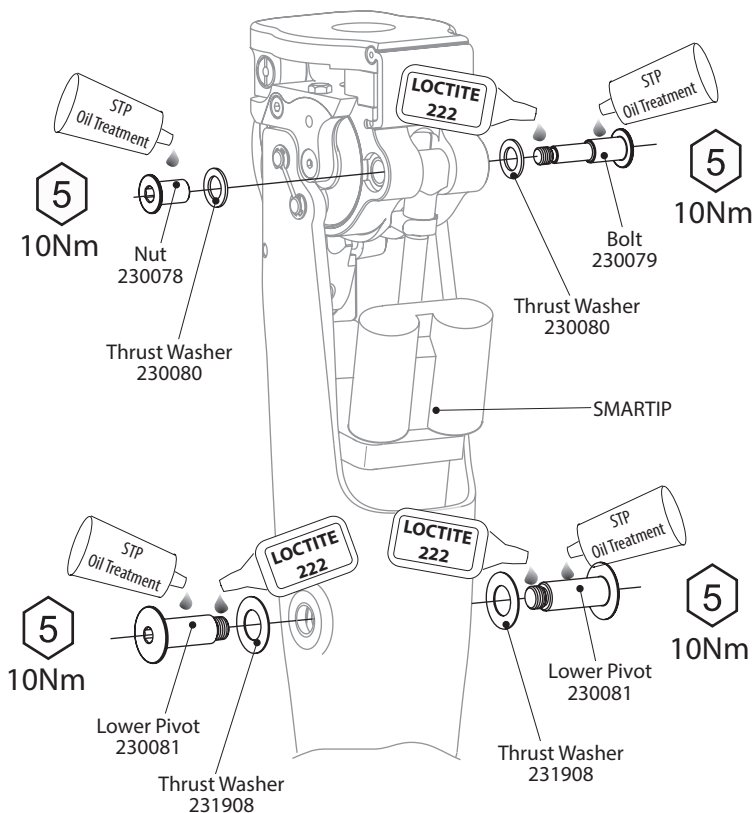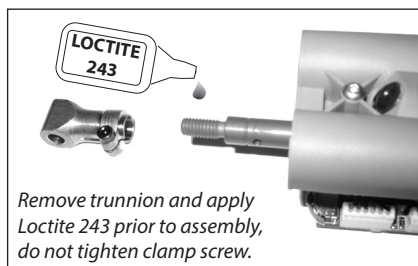

SMARTIP cylinder attachment to ESK+Demountable Shin



(Fitting Kit 019668)

The ESK+ extension stop should be wound fully home. No adjustment is necessary.



User/attendant should be aware of potential finger trap hazards at all times.

SMARTIPSF30
SMARTIPSF30PYR
SMARTIP4B30
SMARTIP4B30PYR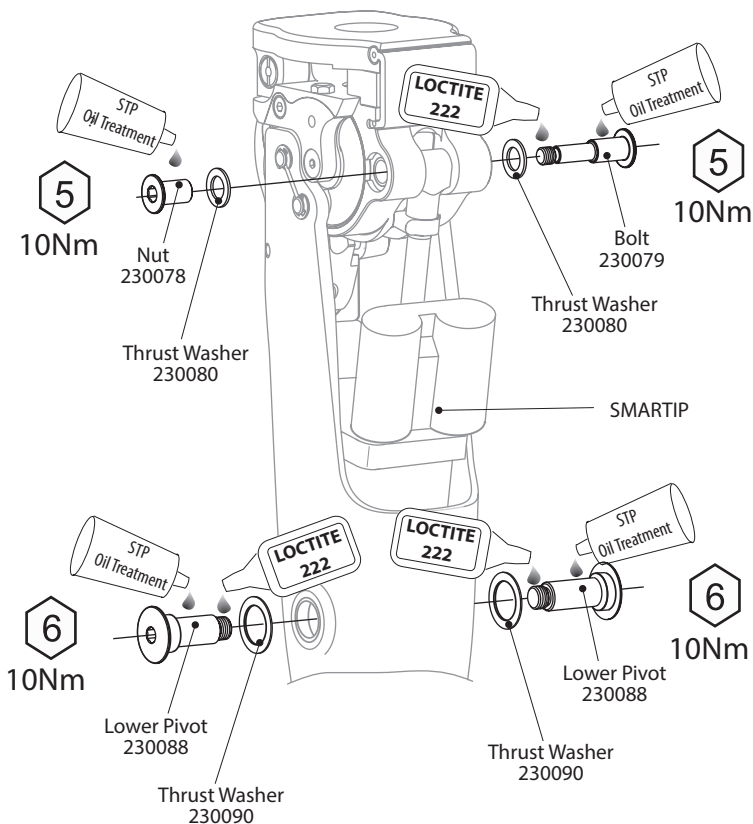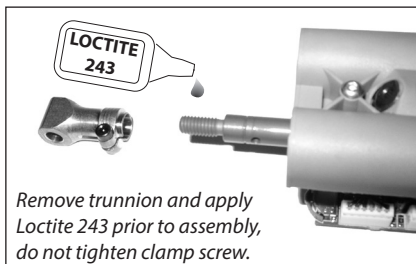

Length Adjustment

For modules supplied with control cylinder already assembled into the shin, the following has already been carried out.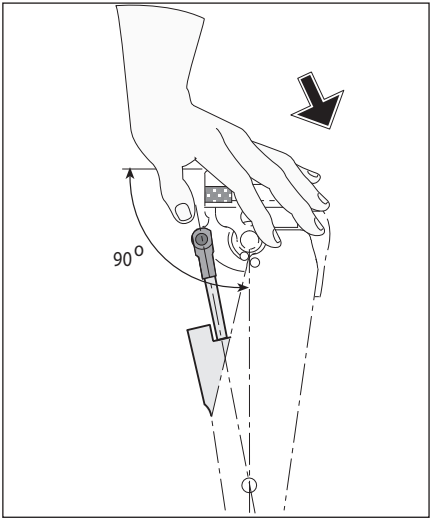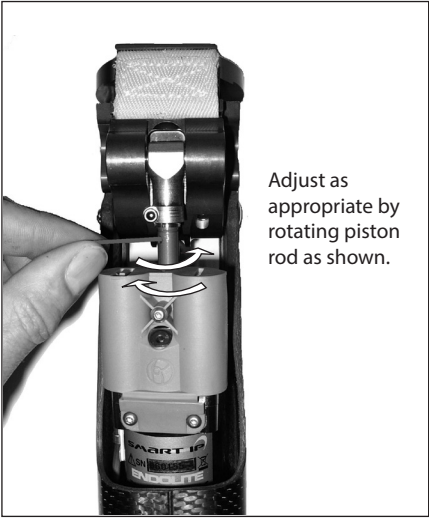

Battery/Fuse Renewal

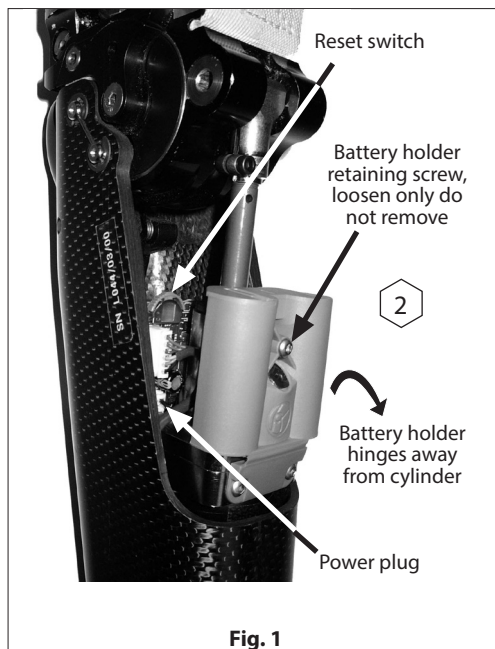
Fig. 1

Battery Removal/Renewal (Fig.1)

When the batteries begin to fail a warning is given in the form of ten short beeps being emitted from the limb. The limb will still be usable, and you should seek to replace them at the earliest opportunity. Upon complete battery failure, the limb will emit one long beep. The limb is still safe to use as a medium setting will have been selected, though speed changes will not now occur.

1. Loosen battery holder central retaining screw (2mm A/F Wrench Key).
2. Hinge holder away from cylinder.
3. To remove batteries, depress spring terminal at negative end of each battery and pull out positive end.
4. On refitting, ensure batteries and terminals are clean and note correct polarity as shown inside holder.
5. Hinge holder back to cylinder and tighten retaining screw.
6. Ensure power plug is connected.
7. Press reset switch once, a beep will confirm satisfactory reset.
8. Dispose of used batteries in accordance with manufacturers recommendations.

Battery type: CR123A-3V Lithium Camera Battery

NOTE

Always replace batteries in pairs.

Do not store batteries in sunlight, near heat or in moist conditions.

Should unit require return to Blatchford, please remove and dispose of the batteries. (Disposal as per local waste handling regulations).

Replacing the fuse (Fig 1)

- Loosen the battery holder retaining screw.
- Open battery holder by hinging away from cylinder.
- Disconnect power plug from PCB.
- Using a suitable thin tool 'hook' the fuse from it's holder and discard. Replace the fuse with new (950043).
- Close battery holder and tighten retaining screw.
- Reconnect power plug.

Manufacturer's Registered Address

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Blatchford SAS
Endolite India Ltd