move/active/xplore Liner move/xplore Sleeve

Instructions for Use

BM(C/L)U(16-44)-(3/6)

BA(C/L)U(16-44)-(3/6)

BX(C/L)U(16-44)-(3/6)

BMSU(16-44)-3

BXSU(20-32)-2

EN	Instructions for Use	2
FR	Instructions d'utilisation	11
DE	Gebrauchsanweisung	20
IT	Istruzioni per l'uso	29
ES	Instrucciones de uso	38
NL	Gebruiksaanwijzing	47

Contents

Contents	2
List of Products Covered by this Instructions for Use	3
1 Description and Intended Purpose	4
2 Warnings.....	4
3 Sizing and Fitting.....	6
4 Maintenance	7
5 Frequently Asked Questions and Possible Solutions	9

List of Products Covered by this Instructions for Use

Thermoplastic Elastomer (TPE) Liners

Name	Part Number
Blatchford move Liner	BM(C/L)U(16-44)-(3/6)
Blatchford active Liner	BA(C/L)U(16-44)-(3/6)
Blatchford xplore Liner	BX(C/L)U(16-44)-(3/6)

Thermoplastic Elastomer (TPE) Sleeves

Name	Part Number
Blatchford move Sleeve	BMSU(16-44)-3
Blatchford xplore Sleeve	BXSU(20-32)-2

Part Number Information

	Brand	Type	Type	Shape	Size	Thickness
Name Breakdown	Blatchford	move, active, or xplore	Cushion, Locking, or Sleeve	Uniform	Size	Thickness
Part No.	B	M, A, or X	C, L, or S	U	16-44	2, 3 or 6

1 Description and Intended Purpose

Intended Purpose

- Blatchford move/active/xplore Liners and Sleeves are intended to be used as part of a lower limb external prosthetic leg.
- The liner is worn on the residual limb to provide cushioning and protection.
- The sleeve helps maintain the suspension of the prosthetic leg by creating an airtight seal at the top of the prosthetic socket. The sleeve is used in combination with a cushion liner.
- Locking liners have a distal attachment for installation of the pin and are used in combination with a locking system to allow the suspension of the prosthetic leg.

Limitations and Contraindications

- Do not wear on injured skin or open wounds. Protect the injured area with a suitable device before wearing the liner/sleeve.
- The material is resistant to salt and chlorinated water. Consult a certified professional for using a prosthetic leg in water environment.
- The material may deteriorate if exposed to high temperature for a prolonged time. Please refer to *Warnings* and *Storing the Liner/Sleeve sections*.

2 Warnings

 This warning symbol highlights important safety information which must be followed carefully.

Skin Rashes and Irritations

-  To prevent skin irritations, it is important to clean the residual limb and the liner/sleeve every day.
-  After washing, rinse the surface of the residual limb to remove all traces of soap, which could cause rashes and irritations. Make sure the residual limb is completely dry before donning the prosthesis.
-  Please refer to *Cleaning the Residual Limb* and *Cleaning the Liner/Sleeve sections*
If skin rashes or irritations occur, discontinue the use and consult a certified prosthetist.

Deterioration of the Material

-  The material will delaminate if it comes into contact with oily substances. Avoid petroleum jelly and other oil-based lubricant on the skin. Wash the liner/sleeve with a hypoallergenic pH-balanced soap; do not use oil-based soap. Do not soak in acetone or you will dissolve the thermoplastic elastomer (TPE).
-  Please refer to *Cleaning the Residual Limb*, *Cleaning the Liner/Sleeve* and *Tips to Keep the Product in Good Condition* sections.
-  The material may deteriorate if exposed to high temperature for a prolonged time. When not in use, keep the liner away from direct source of heats; store in a cool area.
-  Lying flat for long periods may cause creases to develop: always keep the form insert in the liner/sleeve when not in use. Do not leave the liner/sleeve semi-inverted. When the prosthesis is not in use, separate the liner/sleeve from the socket.
-  If liner or sleeve is pulled too hard, the fabric will separate from the TPE.
-  Please refer to *Storing the Liner/Sleeve* and *Tips to Keep the Product in Good Condition* sections.

Integrity of the Product

-  Discontinue the use of the liner/sleeve if the material shows signs of creases, holes and degradation, and consult a certified prosthetist.
-  Holes in the sleeve may cause loss of suspension. Inspect the sleeve daily to verify there are no holes; discontinue use and consult a certified prosthetist if any holes have been found.

Donning the Liner/Sleeve

-  Remove any air pockets trapped between residual limb and the liner, to prevent pain.
-  If using a liner with a distal attachment (pin), check that the pin is properly aligned. Check that the locking mechanism is safely engaged before starting to walk with the prosthetic leg.
-  Please refer to *Donning Liners* and *Donning Sleeves* sections for a detailed explanation of the correct procedure.

3 Sizing and Fitting

Sizing Liners

- Measure the circumference 6 cm from the apex of the residual limb.
- Select a size inferior to the measured circumference
- Apply a reduction rate of 3-5 % to the measured limb size to calculate advised liner size.

Donning Liners

- Make sure the residual limb is completely dry.
- Invert the liner to fit so it firmly adheres to the surface of the distal end of the residual limb.
- Roll it up slowly to prevent air pockets from forming during this stage.
- Remove any air pockets trapped between limb and liner.
- If using a liner with a distal attachment (pin), check that the pin is properly aligned.
- For residual limb volume loss, a sheath, sock or alternative volume management solution may be used.

Sizing Sleeves

- Measure the circumference 10 cm above patella of the residual limb.

Donning Sleeves

- Invert the sleeve so that the inside is facing out and don onto the socket.
- Introduce the residual limb into the socket.
- Roll the sleeve onto the proximal end of the residual limb.
- For best results, verify the sleeve is placed properly.

Note... For our complete product catalogue and more information about specific sizing, please use Blatchford's Socket Technology Catalogue which can be downloaded from the website www.blatchfordmobility.com

4 Maintenance

Cleaning the Residual Limb

- Wash the residual limb every evening with warm water and a pH-balanced cleanser.
- After washing, rinse the surface of the residual limb to remove all traces of soap, which could cause rashes and irritations.
- Dry the residual limb with a clean towel. If washed in the morning, make sure that the residual limb is completely dry before donning the prosthesis.
- Avoid using petroleum based products and other oil based lubricants on skin.

Cleaning the Liner/Sleeve

 **Do not use oil based soap.**

- **Daily:** Hand wash the surface with warm water and hypoallergenic pH-balanced soap.
- Use a wash cloth to wipe the surfaces but do not rub/scrub or you may damage the product.
- Rinse well after washing.
- Wipe dry with a clean, dry towel. Allow the product to dry thoroughly before use and do not dry on sources of direct heat.
- **Weekly:** Pour a small amount of rubbing alcohol on a soft, clean cloth. Wipe the liner/sleeve with the cloth; dry thoroughly with a towel. Do not soak the liner/sleeve in alcohol.
- To remove a stain on the fabric side, use a cloth dampened in acetone (nail polish remover) and rub until clean.

 **Do not soak in acetone or you will dissolve the TPE.**

Storing the Liner/Sleeve

- Always keep the foam insert in the liner/sleeve when not in use: lying flat for long periods may cause creases to develop.
- Do not leave the liner/sleeve semi-inverted.
- When the prosthesis is not in use, you should separate the liner/sleeve from the socket.

Tips to Keep the Product in Good Condition

- Avoid using oil based products. The TPE will delaminate if it comes into contact with oily substances. If the patient uses suntan lotion or other oil-based creams, the residual limb must be washed with a pH-balanced cleanser and rinsed thoroughly before putting the liner back on the limb.
- Should you need to use ointments of any sort, they should be applied when the prosthesis is not in use. Wash and rinse the residual limb thoroughly before putting the liner back on.

5 Frequently Asked Questions and Possible Solutions

Question: My patient is using a Blatchford move/active/xplore liner/sleeve for the first time and complained about excessive perspiration, is this normal?

Answer: Excessive perspiration is normal. It will gradually diminish over a period of two to six weeks. If excessive perspiration continues, consider whether a liner designed for heat dissipation such as Blatchford Silcare Breathe Liners would be more appropriate.

Question: Do I trim the liner above the knee, just above the socket edge, or do I reflect the liner over the socket to provide suspension?

Answer: You can do all of the above. We recommend that you trim 2.5 cm (1 ") above the edge of the socket. To trim, you may use scissors, a pizza cutter or a hot knife for plastics. If you choose to fold the liner over the socket, make sure that the edge of the socket is well taped or a hole may develop in the liner (in such a case, the warranty will not apply.) For additional security, a small drop of instant adhesive on the seam directly below the trimmed edge may be used to help prevent the seam from coming undone.

Question: Do stains affect the performance of the liner/sleeve?

Answer: No, performance is not affected. If the stain cannot be removed with the procedures described in the Cleaning Section, it is recommended that the exposed portion of the liner/sleeve be covered with a 35-40 cm (14-16 ") piece of stockinet.

Question: What is the cause of micro cracking on the surface of my liner/sleeve?

Answer: Micro cracking can take place either when:

- a) Petroleum products or organic oils (including human fatty acids) come in prolonged contact with the liner/sleeve. Some patients' skin may be oily enough to induce micro cracking. In such cases the life expectancy of the liner is greatly reduced. The prosthetist will decide if the liner is appropriate for such patients.
- b) The liner/sleeve is stretched over 100% of its original dimension and kept in a stretched position over a long period of time.

Question: How do I prevent the pin from unscrewing from the locking liner's distal end?

Answer: Use Loctite type thread locking compound 242 or equivalent.

Question: Do Blatchford move/active/xplore liners or sleeves contain latex?

Answer: No, all of Blatchford move/active/xplore liners and sleeves are latex free.

Liability

The manufacturer recommends using the device only under the specified conditions and for the intended purposes. The device must be maintained according to the instructions for use supplied with the device. The manufacturer is not liable for any adverse outcome caused by any component combinations that were not authorized by them.

CE Conformity

This product meets the requirements of the European Regulation EU 2017/745 for medical devices.

Medical Device

Single Patient – multiple use

Compatibility

Combination with Blatchford branded products is approved based on testing in accordance with relevant standards and the MDR including structural test, dimensional compatibility and monitored field performance.

Combination with alternative CE marked products must be carried out in view of a documented local risk assessment carried out by a Practitioner.

Warranty

Liners are warranted for 12 months.

Sleeves are warranted for 3 months.

The user should be aware that changes or modifications not expressly approved could void the warranty, operating licenses, and exemptions.

See Blatchford website for the current full warranty statement.

Reporting of Serious Incidents

In the unlikely event of a serious incident occurring in relation to this device it should be reported to the manufacturer and your national competent authority.

Environmental Aspects

Where possible the components should be recycled in accordance with local waste handling regulations.

Retaining the Packaging Label

You are advised to keep the packaging label as a record of the device supplied.

Trademark Acknowledgements

Blatchford is a registered trademark of Blatchford Products Limited.

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Endolite India Ltd.
Blatchford Inc.
Blatchford Europe GmbH
Blatchford Products Ltd.