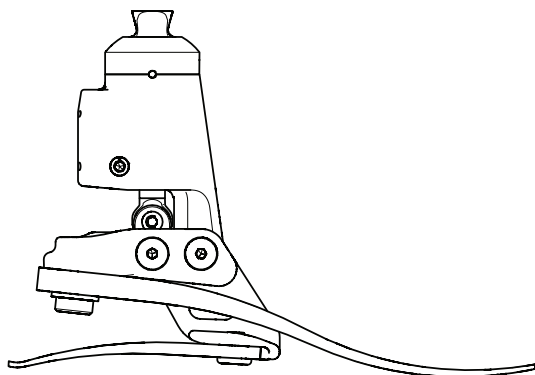
# EchelonER

## User Guide

ECER22L1S–ECER30R8S  
ECER22L1SD–ECER30R8SD

|    |                          |    |
|----|--------------------------|----|
| EN | User Guide               | 2  |
| DA | Brugervejledning         | 8  |
| NO | Brukerhåndbok            | 14 |
| FI | Käyttöopas               | 20 |
| SV | Användarhandbok          | 26 |
| RU | Руководство пользователя | 32 |
| ZH | 用户指南                     | 38 |
| AR | دليل المستخدم            | 44 |

ECER22L1S–ECER30R8S

ECER22L1SD–ECER30R8SD

Please keep these instructions.

Contents

Contents .....2

1 Description and Intended Purpose .....2

2 Safety Information .....3

3 Construction .....4

4 Maintenance .....5

5 Limitations on Use .....5

6 Technical Data .....6

1

Description and Intended Purpose

The term *device* is used throughout this document to refer to EchelonER.

Application

This device is to be used exclusively as part of a lower limb prosthesis.  
Intended for single user.

The device provides limited self-alignment of the prosthesis on varied terrain and following footwear changes. It is intended to improve on postural sway and symmetry while easing abnormal pressures at the socket interface.

This device provides limited self-alignment of the prosthesis on varied terrain and allows the ankle to adjust to a moderate variation in heel heights. It is intended to improve on postural sway and symmetry while easing abnormal pressures at the socket interface.

A moderate-energy-return foot with multi-axial ankle movement. Independent heel and toe springs provides some axial deflection. The split toe provides good ground compliance.

## 2 Safety Information

This warning symbol highlights important safety information which must be followed carefully.

Ensure only suitably retrofitted vehicles are used when driving. All persons are required to observe their respective driving laws when operating motor vehicles.

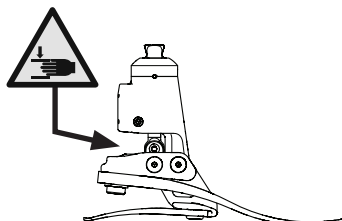
Do not ascend /descend ladders, the added range of DF/PF motion may cause the foot to slip.

Due to the amount of ankle movement available with this device, take extra care when operating any form of machinery.

Be aware of finger trap hazard at all times. It is recommended that a flexible cosmetic cover is used to minimise risk.

Always use a hand rail when descending stairs and at any other time if available.

Following activities which result in a high degree of self alignment e.g. after sitting or stairs descent, weight should be applied in a neutral position to restore the foot to its optimal standing position.

Any changes in the performance or function of the limb e.g. restricted movement, non-smooth motion or unusual noises should be immediately reported to your service provider.

The device is not suitable for extreme sports, running or cycle racing, ice and snow sports, extreme slopes and steps. Any such activities undertaken are done so completely at the users' own risk. Recreational cycling is acceptable.

Assembly, maintenance and repair of the device must only be carried out by a suitably qualified clinician.

To minimise the risk of slipping and tripping, appropriate footwear that fits securely onto the footshell must be used at all times.

After continuous use the ankle casing may become hot to the touch.

Avoid exposure to extreme heat and/or cold, which may affect ankle stiffness.

The user must not adjust or tamper with the setup of the device.


The user should contact their clinician if their condition changes.

### 3 Construction
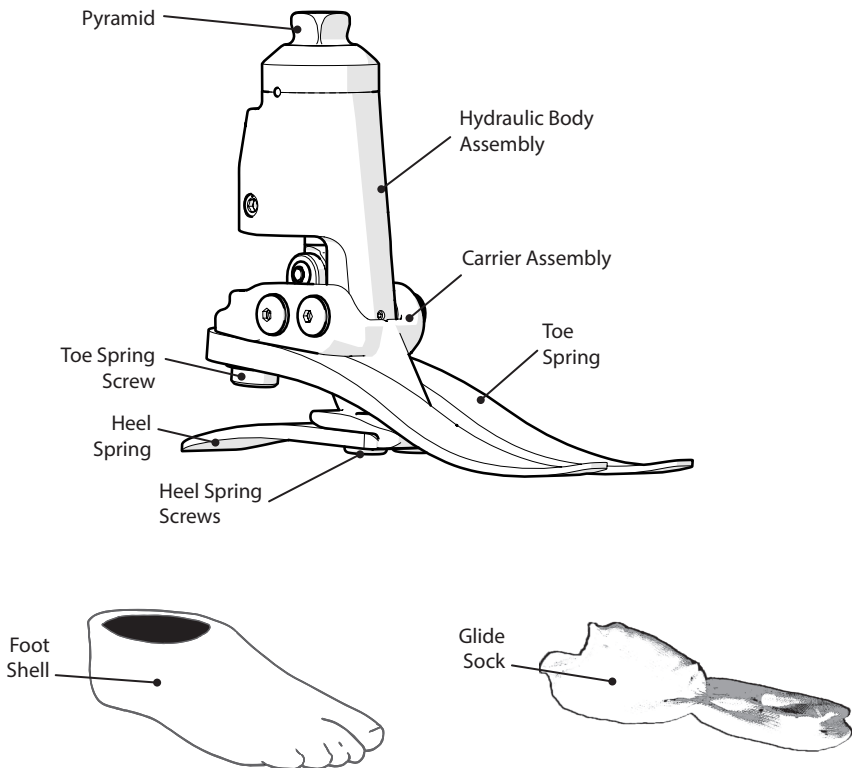
#### Principal Parts:

- Hydraulic Body Assembly including pyramid (aluminum/St. Stl./titanium)
- Carrier Assembly (aluminum/St. Stl.)
- Heel & Toe Springs (e-carbon)
- Spring Attachment Screws (titanium/St. Stl.)
- Glide Sock (UHM PE)
- Foot Shell (PU)

---

## 4 Maintenance

Maintenance must be carried out by competent personnel.

We recommend a regular visual check at least annually. Signs of wear that may affect function should be reported to the service provider (e.g. significant wear or excessive discoloration from long term exposure to UV).

Any changes in performance of this device must be reported to the practitioner.

Changes in performance may include:

- Increase in ankle stiffness
- Reduced ankle support (free movement)
- Any unusual noise

The practitioner must also be informed of any changes in body weight and/or activity level.

### Cleaning

Use a damp cloth and mild soap to clean outside surfaces. Do not use aggressive cleansers.

---

## 5 Limitations on Use

### Intended Life

A local risk assessment should be carried out based upon activity and usage.

### Lifting Loads

User weight and activity is governed by the stated limits.

Load carrying by the user should be based on a local risk assessment.

### Environment

This device is waterproof to a maximum depth of 1 meter.

Thoroughly rinse with fresh water after use in abrasive environments such as those that may contain sand or grit, for example, to prevent wear or damage to moving parts.

Thoroughly rinse with fresh water after use in salt or chlorinated water.

Foot products must be adequately finished to prevent water ingress into the foot shell where possible. If water enters the foot shell, the limb should be inverted and dried before further use.

It is recommended that only Blatchford products be used in conjunction with the device.

Exclusively for use between -15°C and 50°C  
(5°F to 122°F).

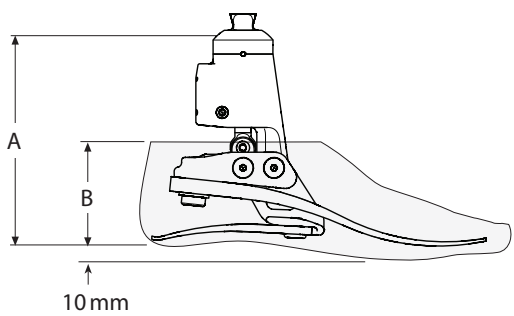
Suitable for submersion

# 6 Technical Data

|                                                                                                           |                                                                      |
|-----------------------------------------------------------------------------------------------------------|----------------------------------------------------------------------|
| Operating and Storage Temperature Range:                                                                  | -15 °C to 50 °C<br>(5 °F to 122 °F)                                  |
| Component Weight (Size 26N)                                                                               | 990 g (2.2 lb)                                                       |
| Activity Level:                                                                                           | 3                                                                    |
| Maximum User Weight:                                                                                      | 125 kg (275 lb)                                                      |
| Proximal Alignment Attachment:                                                                            | Male Pyramid (Blatchford)                                            |
| Range of Hydraulic Ankle Motion<br>(excludes additional range of motion provided by heel and toe springs) | 19 degrees plantar flexion<br>to 6 degrees dorsiflexion              |
| Build Height:<br>(See diagram below)                                                                      | (sizes 22–24) 142 mm<br>(sizes 25–26) 147 mm<br>(sizes 27–30) 152 mm |
| Heel Height                                                                                               | 10 mm                                                                |

## Fitting Length

| Size  | A      |
|-------|--------|
| 22–24 | 142 mm |
| 25–26 | 147 mm |
| 27–30 | 152 mm |

| Size  | B     |
|-------|-------|
| 22–26 | 65 mm |
| 27–28 | 70 mm |
| 29–30 | 75 mm |

## Liability

The manufacturer recommends using the device only under the specified conditions and for the intended purposes. The device must be maintained according to the instructions for use supplied with the device. The manufacturer is not liable for any adverse outcome caused by any component combinations that were not authorized by them.

## CE Conformity

This product meets the requirements of the European Regulation EU 2017/745 for medical devices. This product has been classified as a class I device according to the classification rules outlined in Annex VIII of the regulation. The EU declaration of conformity certificate is available at the following internet address: [www.blatchfordmobility.com](http://www.blatchfordmobility.com)

Medical Device

Single Patient – multiple use

## Warranty

The device is warranted for 36 months - foot shell 12 months - glide sock 3 months. The user should be aware that changes or modifications not expressly approved could void the warranty, operating licenses and exemptions. See Blatchford website for the current full warranty statement.

## Reporting of Serious Incidents

In the unlikely event of a serious incident occurring in relation to this device it should be reported to the manufacturer and your national competent authority.

## Environmental Aspects

Where possible the components should be recycled in accordance with local waste handling regulations.

## Trademark Acknowledgements

EchelonER and Blatchford are registered trademarks of Blatchford Products Limited.

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**Blatchford Inc.**
Blatchford Europe GmbH
Blatchford Products Ltd.